



## SIGHTSTAND SETUP & USE INSTRUCTIONS



1. Identify magsafe ring on phone, or attach provided metal magsafe sticker



2. Attach blue rubber magnet side of SightStand to phone.



3. Pull open top leg of SightStand from inside, with fingertip



4. Activate phone screen app menu and search for magnifier - download free magnifier app if not present. Iphone - swipe down on home screen and type magnifier. Android - Go to setting, search "magnifier. Turn on magnifer shortcut, then set to "floating over other apps"



5. Rest stand legs on reading material to read



6. If screen is out of focus, open lens clip provided (see back side)





## LENS CLIP INSTRUCTIONS



1. Remove small cover on lens, and screw into clip



2. Remove large cover on lens, and screw into clip



3. Use finger to cover phone camera and find which camera your phone uses for the magnifier



4. Squeeze open clip, to fit around phone, with lens on the phone camera with your finger.



5. Adjust fitment of lens clip and keep in place for reading.

